

See now. Know now.™

Endosee® Advance Direct Visualization System Office Checklist*



Supplies for Procedure Using Endosee Advance

- 1 Cleaned/disinfected Endosee Advance display module
- 1 Sterile single-use Endosee Advance cannula
- Sterile saline (hanging bag with pressure cuff or 60cc syringe and extension tubing)
- Endosee Convenience Kit†
 - 1 Drape (36"x36")
 - 3 Swabsticks (8")
 - 1 Fundus Sound
 - 1 Cervical Dilator
 - 1 Syringe (60cc) with Extension Tubing
 - 1 Under-Buttocks Drape Pouch
 - 1 Lubricating Jelly (3g)
 - 1 Povidone-Iodine Solution (3/4 oz)
- Speculum (open-sided speculum recommended)
- Disposable Os Finder™
- Alcohol wipes and/or CaviWipes®

Optional

- Surgical Stabilizer or Tenaculum
- Working Channel Instruments
- EM Biopsy Device packaged with Endosee Advance cannula
- EM Biopsy Device collection container



Other Considerations

- Patient's pain tolerance and level of anxiety – NSAIDs, Topical Analgesic, Anxiolytics
- Patient's parity/cervical status – Dilation, Cervical softening/Cytotec®
- Patient's time of cycle, if premenopausal



Endosee Advance Steps

- 1 Turn Endosee Advance display module "ON," press "START," press "NEW PATIENT," and enter a patient ID. Press "START EXAM".
- 2 Open the sterile single-use cannula package.
- 3 Open the cannula handle to the down position and slide on the display module, locking securely into place.
- 4 Check that the light is on the cannula.
- 5 Attach saline tubing to the cannula. Flush the system until all air bubbles in the fluid line are purged to remove air. The system is now ready for use.
- 6 When case is completed, you can remove the display module to review any recorded photos or video with your patient.
- 7 Dispose of the single-use cannula per your facility's policy.
- 8 Clean/disinfect display module as per Instructions For Use.
- 9 Slide display module directly onto the Endosee Advance docking station after each use to recharge battery.
- 10 Connect docking station via USB cable to your computer to transfer any photos or video.

Part Number	Description
ES9000	Endosee Advance Display Module
ESPX5	Endosee Advance Cannula, 5-pack
ES-TRAY	Endosee System Convenience Kit, 5-pack
ES-BPSY	Biopsy Forceps, 5 Fr, 455 mm
ES-LNGR	Spoon Forceps, Long Serrated, 5 Fr, 455 mm
ES-FBGR	Foreign Body Grasper, Fenestrated, 5 Fr, 455 mm
ES-SCIS	Ureter Slitting Scissors, Single Action, 5 Fr, 455 mm
ES-DISP-AGR	Disposable Alligator Grasper Forceps , 5 Fr, 460 mm, 5-Pack

*Not a substitute for the Endosee Advance Instructions for Use.

†Sold separately.